SURFACE DISINFECTION - CaviCide

IF GROSS DEBRIS IS PRESENT ON THE SURFACE:

CLEAN BEFORE DISINFECTING



1. Spray CaviCide1 directly onto surface to be cleaned.



DISINFECT AFTER CLEANING



4. Spray CaviCide1 directly onto precleaned surface, thoroughly wetting area. Leave surface wet for one (1) minute.



6. Discard used towelette.

IF <u>NO VISIBLE</u> GROSS DEBRIS IS PRESENT ON THE SURFACE:

ONE STEP CLEANING AND DISINFECTING



1. Spray CaviCide1 directly onto surface, thoroughly wetting area. Leave surface wet for one (1) minute.



2. Wipe with paper towel.



3. Discard used towel.

PRODUCT INFORMATION		
PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-5002	CaviCide1 - 2 ounce spray	48 bottles/case
13-5008	CaviCide1 - 8 ounce spray	12 bottles/case
13-5024/11-5024	CaviCide1 - 24 ounce spray	12 bottles/case
13-5000/11-5001	CaviCide1 - 1 gallon	4 bottles/case
13-5025/11-5025	CaviCide1 - 2.5 gallon	2 bottles/case
13-5055/11-1055	CaviCide1 - 55 gallons	1 drum

Note: Follow standard precautions for personal protection as appropriate.



CaviCide1 Family of surface disinfectants



Lit #77-1095-

For more information, visit metrex.com/education, or call 800.841.1428.